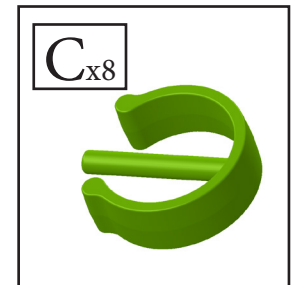
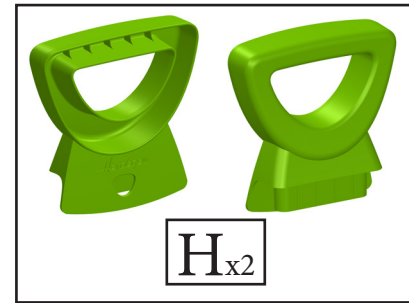
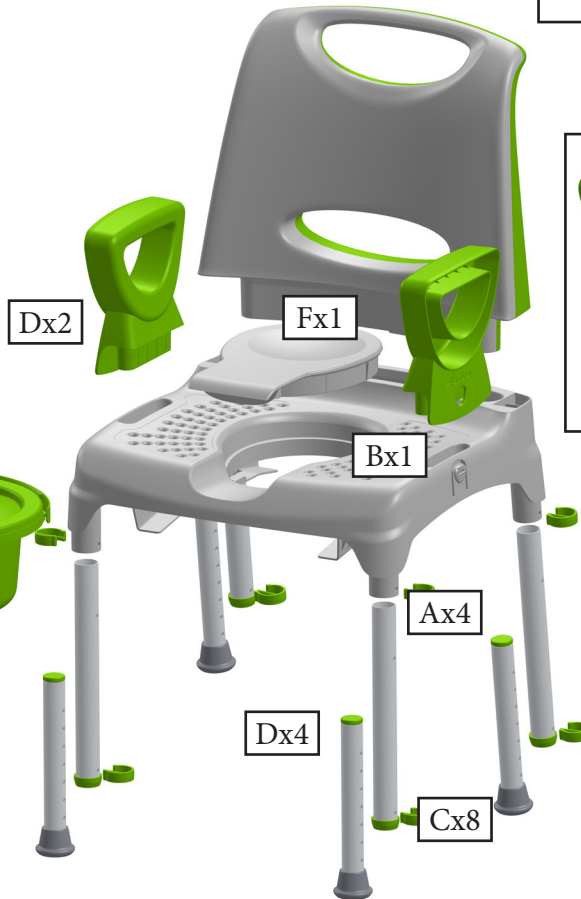
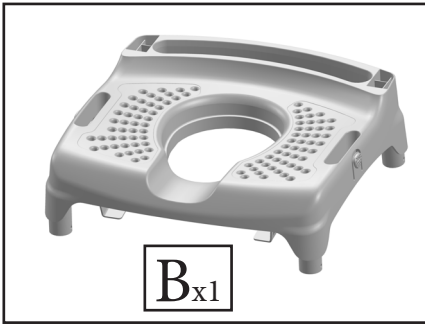
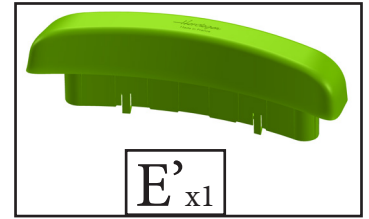
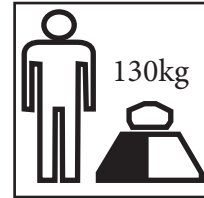
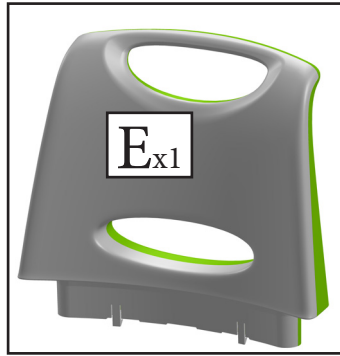
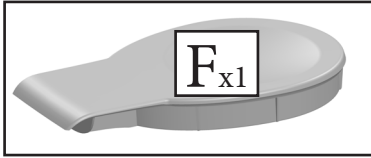
1


1'


Ax4

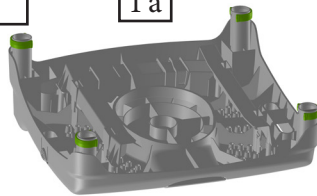
1'a

1'b





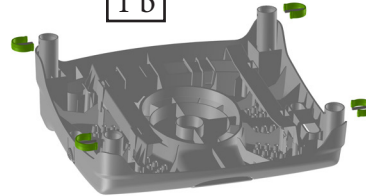
1'c

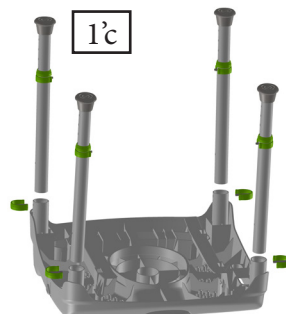1'd

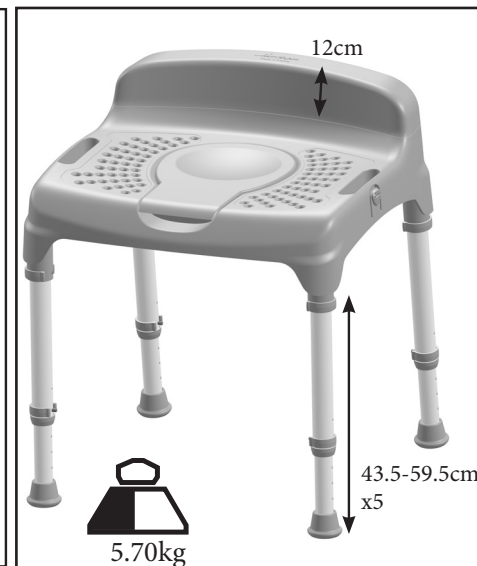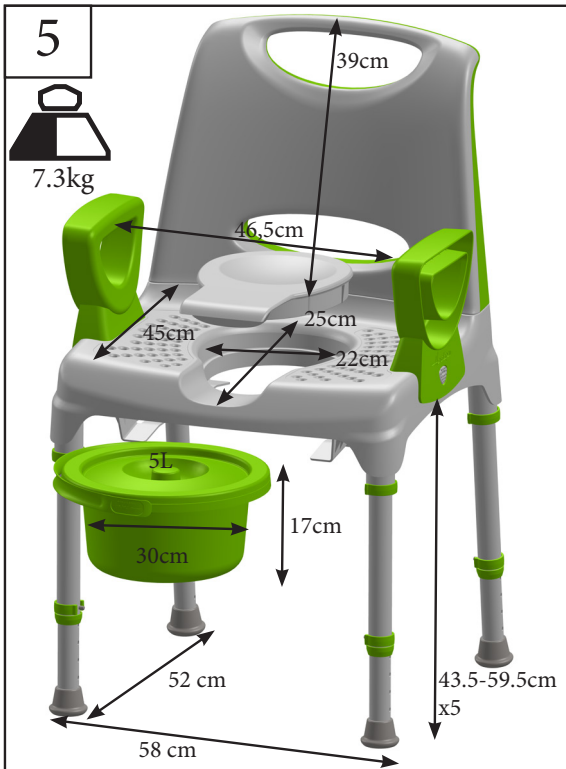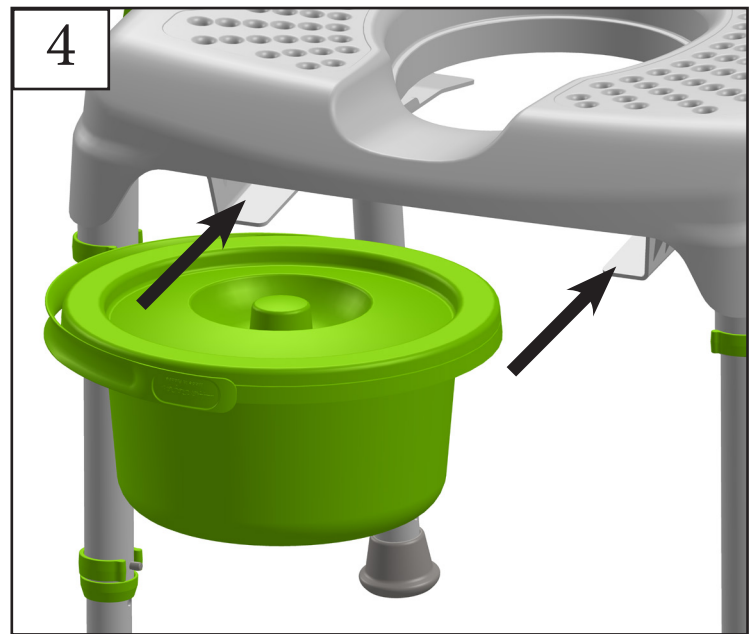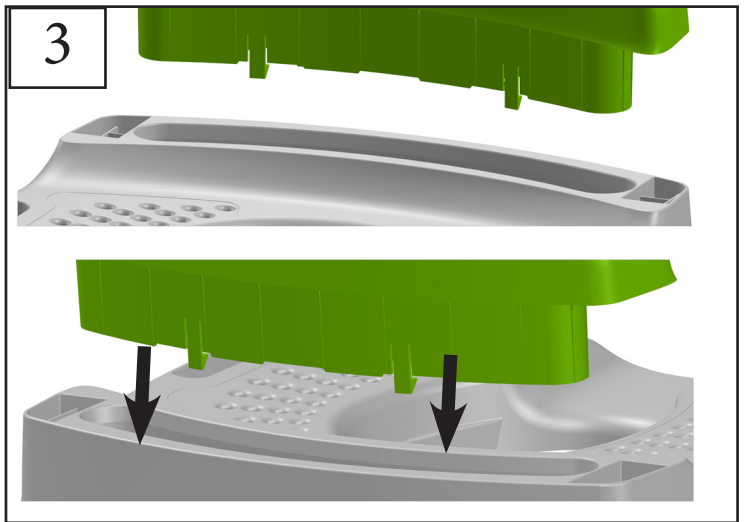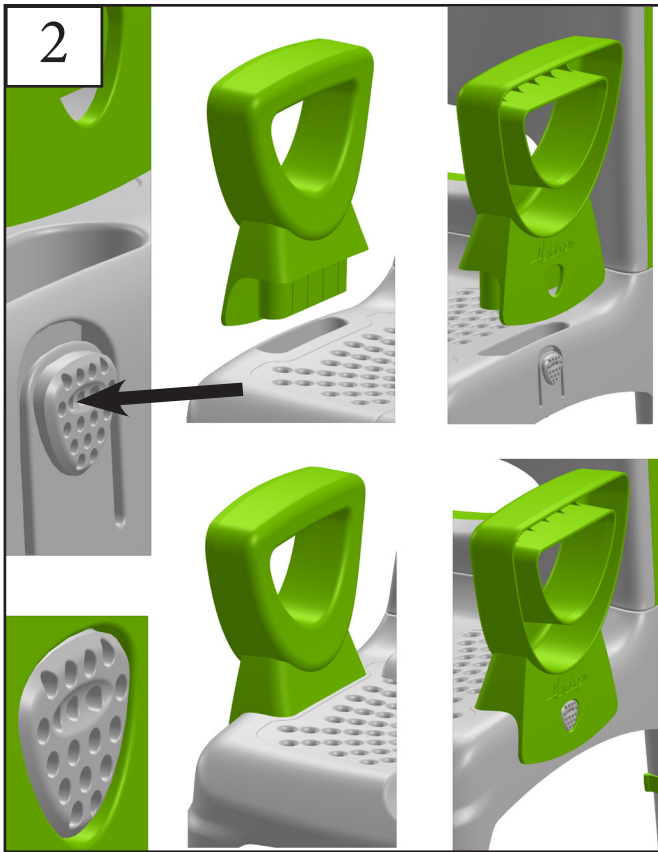
540330 AQ-TICA Confort Anis
540331 AQ-TICA Confort Grey

540332 AQ-TICA Club Anis
540333 AQ-TICA Club Grey

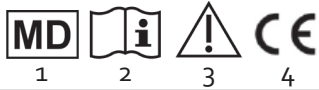54031 AQ-TICA Essential

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EN- Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (tubes, tips, armrests, backrest).
- ♥ Use only as a commode-shower chair and in proper way: don't stand up on the chair.
- ♥ ALWAYS observe the weight limit on the labeling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After assembling the chair, make sure that it is securely locked in position and level to the ground before use.
- ♥ DO NOT hang anything on the chair.
- ♥ ALWAYS test to make sure that the chair and attachments are properly and securely locked in place before use.
- ♥ In case you didn't use your chair for a long time, let it checked by the technical staff, before use.
- ♥ Do not store the product outside, it must be used at home.
- ♥ Do not install or remove the pan while the patient is sitting on the chair! High risk of sexual injury.
- ♥ Be careful when removing the armrests to avoid trapping your fingers.
- ♥ Check the chair before use, in case of doubt ask your physician or provider.
- ♥ Use only the chair on a flat and solid ground.
- ♥ It is forbidden to sit on the armrests.
- ♥ Do not lean out of the chair to avoid losing balance.
- ♥ Do not use the chair alone but with assistance.
- ♥ Do not let children play with the chair.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.
- ♥ The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. Intended use, Indication and Contraindication

The shower-commode chairs AQ-TICA can be used at home or in nursing homes to assist a person who has difficulty getting to the bathroom by bringing the toilet into the bedroom or enabling the person to sit while showering.

Indication

Used in the home when the patient is too debilitated to reach the bathroom.

Contraindication

Don't use the commode chair in case of:

- Cognitive disorder - Loss of balance - Sitting impairment

3. Installation

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again. The chair can be used on toilet too, remove the pan first. The chair can be used over a toilet bowl: make sure you have removed the pan before use. The chair can be used in the shower and under water.

Beware ! The plastic shell seat has holes to drain water away from the seat. In very few cases, those drainage holes have the potential to entrap genitalia or rectal prolapse, leading to pain and serious injury such as cuts, lacerations and testicular bruising. If in doubt, please consult with your physician or retailer to determine a safe use of the device. You can also place material such as a towel over bath seats to prevent flesh or anatomy passing through.

Supplied Parts

See Fig.01

Installation

- 1-Make sure that all the components of the chair are present and that none of them have been damaged. (Different backrests are delivered depending on the model chosen, see [E] or [E']). The legs are delivered ready to mount).
- 2- Remove the clips [C] from the seat [B], see figures (1'a) and (1'b)
- 3-Slide the legs [A] into the holes of the seat [B], see figure (1'c) and lock them with the safety clips [C], see figure (1'd).
- 4-Slide the armrests [H] into the seat [B] until you hear a click. Push on the button to remove the armrests [D], see figure (2).
- 5-Slide the backrest [E] or [E'] into the seat [B], see figure (3).
- 6-Place the lid [F] on the seat [B] in order to close the hole in the seat.
- 7-Slide the commode bucket [G] under the seat [B] using the holding rails, see figure (4).

Leg adjustment:

Remove the clip and adjust the leg extension to the required height adjustment hole then place the clip again. Make sure that the clip is fully engaged through the extension leg hole. Check the stability before use. Adjust all legs at the same level.

Rental:

This product can be rented to different patients. A thorough maintenance, cleaning and disinfection must be performed before the product is given to a new patient in accordance with the following steps:

- 1- Visual inspection: ensure that the product has not been damaged and check for any defect.
 - 2- Functional inspection: ensure that the product functions properly and repair it if needed.
 - 3- Cleaning and disinfection: this product can be cleaned and disinfected with mild, chlorine-free detergents such as Surfanios (Anios laboratories). Do NOT use any abrasive cleaners or high-pressure cleaners that could damage the product.
- Use only appropriate tools to perform the maintenance, cleaning and disinfection. The user's manual is an integral part of the product and must be handed over to every new user.

4. Cleaning

The chair has to be cleaned regularly and after each use with a mild cleaning agent and a soft cloth.

5. Materials

PP (all the plastic parts), aluminium legs and PVC (tips).

6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

7. Technical specifications

This product has a 2-year warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible.

Test : EN-ISO 9999 : 091203, EN 12182, EN 1041, ISO 10993-ff. Duration of use : 2 years.

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