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EN- Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, seatboard...).
- ♥ Check regularly if all screws are tightened.
- ♥ Use only as a bath board and in proper way : don't stand up on the board.
- ♥ ALWAYS observe the weight limit on the labeling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the seat, make sure that it is securely locked.
- ♥ ALWAYS test to see that the seat and attachments are properly and securely locked in place before use.
- ♥ In case you didn't use your bath seat for a long time, let it checked by the technical staff, before use.
- ♥ Do not store outdoor, the product must be used at home.
- ♥ Check the bathboard before use, in case of doubt ask your physician or provider.

- ♥ Do not lean out of the bath seat to avoid losing balance.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Intended use, indications and contraindications

Intended use:

The bath seats make bathing easier and safer. They also make transfers easier. The user seats above the water and can wash safer. The seats are very light and easy to remove and store after usage. The bath seat adjusts easily in width.

Indications

Inability to get in/out of the bath independently and to sit in the bath independently.

Functional impairment or lack of functional capability of the lower extremities
 Severe limited trunk control with enough residual functions of the upper extremities.

Contraindications

Don't use in case of:

- Cognitive disorder - Loss of balance - Sitting impairment - Inadequate hip and knee flexion and inability to transfer safely on / off bath seat independently.

3. Installation

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

A risk assessment must be carried out as to the suitability of this product particularly with new users. The following risks should be considered: Is there a risk of body part or tissue entrapment. Has the product been safely fitted. Is it weight appropriate. Is the board the correct length.

Fitting the handle, see drawings (1) to (4).

To fit the handle, place the Bathboard with cleats facing downwards and introduce the handle to the holes at the end of the board. The handle grip should face towards the centre of the board. From the underside of the Bathboard, screw the large headed white plastic screws into the holes at the ends of the handles. Make sure not to cross the threads. Tighten using a large headed screwdriver or a small coin.

Fitting the bath board

The bathboard handle must be placed opposite to the user's access side.

- 1.Reverse the board and loosen the 4 supports, which hold the cleats in place. Make sure the cleats are free to move and that the non-slip white pads are facing the outside of the board. Not suitable for straight sided baths.
- 2.Slide the 4 cleats back towards the centre of the bath board so that they will easily fit down into the bath.
- 3.Reverse the bath board so that the cleats hang down into the bath and the board is against the wall on the handle side. (Handle should always be on the wall side).
- 4.Position the bath board in the optimum position, at least 2cm of the board must be on the bathtub sides.
- 5.Slide the cleats out so they touch the inside of the bath and tighten the wing screws finger tight (Do not over tighten).
- 6.Reverse the board again so that the cleats are pointing upwards and adjust them outwards by approximately 2mm on each cleat. Now tighten the supports so that they do not move easily.
- 7.Turn the Bathboard up the right way and press it gently down into the bath (Do not force it). If the board slides along the bath, you may need to adjust the supports further outwards. To do this repeat steps 2 to 7.

4. Cleaning

The bath seat has to be cleaned regularly with a mild cleaning agent, hot water and a soft cloth. Dry thoroughly.

5. Materials

The product is made out of plastic materials (polypropylene) and is therefore rust-free.

The box contains:

- Bathboard with 4 anti-slip supports - Handle - User's manual

6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

7. Technical specifications

Duration of use: 5 years

This product has a 2 year warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible.

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